
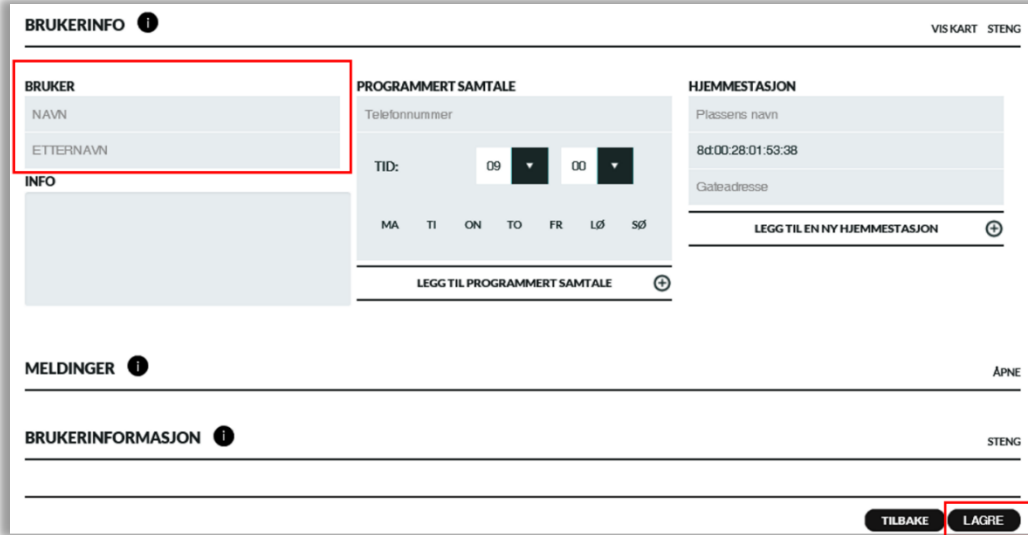
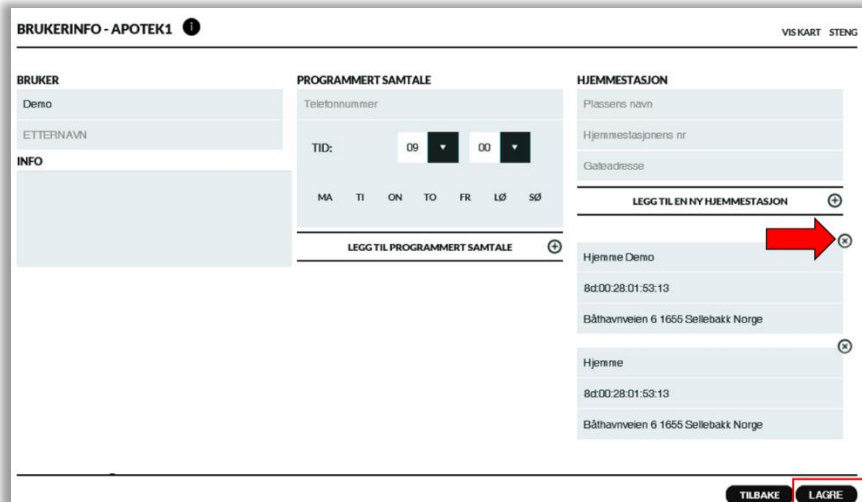


NY BRUKER AV SMARTKLOKKEN: Trykk på ikonet for innstillinger  og fyll inn informasjon om ny bruker av smartklokken.



The screenshot shows the 'BRUKERINFO' settings page. The 'BRUKER' section is highlighted with a red box and contains fields for 'NAVN' and 'ETTERNAVN'. The 'PROGRAMMERT SAMTALE' section includes a 'Telefonnummer' field and a 'TID' selector with dropdowns for '09' and '00', and a weekly schedule 'MA TI ON TO FR LØ SØ'. The 'HJEMMESTASJON' section has fields for 'Plassens navn', 'Bd:00:28:01:53:38', and 'Gateadresse', with a 'LEGG TIL EN NY HJEMMESTASJON' button. At the bottom, there are 'TILBAKE' and 'LAGRE' buttons, with 'LAGRE' highlighted by a red box.

NY ADRESSE: Legg inn ny adresse for hjemmestasjon og slett den gamle adressen.



The screenshot shows the 'BRUKERINFO - APOTEK1' settings page. The 'HJEMMESTASJON' section is highlighted with a red box and contains a list of addresses. A red arrow points to the 'Hjemme Demo' entry, which has a delete icon (X) next to it. The 'LAGRE' button at the bottom is also highlighted with a red box.